



**CellSonic Ltd**



### **Conversion from Storz to CellSonic**

More surgeons are upgrading to CellSonic with its better form of sudden shockwave and wider area of impact. Often they have experience of the Storz Duolith which in comparison they find weak and narrow in beam. They now want to know how to use the CellSonic compared to the Storz and especially what power level to apply. Below is a conversion table which is a suggestion of equivalent effectiveness.

Shockwaves work much better without anaesthetic so either a patient had to suffer or have an anaesthetic and reduced therapy effect. It is known that shockwaves numb the nerves, called the analgesic effect, so the CellSonic Medical machine has Soft Start where the power level gradually increases from a patient tolerable level to a higher level without causing pain. Only CellSonic has this feature.

Storz	CellSonic
.55	10
.5	9
.45	8
.4	7
.35	6
.3	5
.25	4
.2	3
.15	2
.1	1

Al Barsha, POB 450676, Dubai, UAE  
Registered number A004/09/08/0718

[www.cellsonic-medical.com](http://www.cellsonic-medical.com) [www.cellsonic-myshape.com](http://www.cellsonic-myshape.com) [www.cellsonic-beauty.com](http://www.cellsonic-beauty.com)  
[sales@cellsonic-medical.com](mailto:sales@cellsonic-medical.com)